

## Packing Guide for UN3373 Biological Substances Category B by Road using Standard Shipper Boxes

### Step 1

Ensure primary containers are sealed and are clean.

### Step 2

Insert the primary container inside of the 95KPA transport bag with the absorbent material.

### Step 3

Remove the white strip and seal the bag by folding along the black lines.

### Step 4

Make up the outer box as directed and place the sealed bag and documentation inside the box.

### Step 5

Close the box and place your address label with full contact details of the delivery address on the front.

### Step 6

Complete Eurofins Lablink delivery docket and attach to box. The package is now ready for collection.

